
Instruction For Use

Instruction and Precautions:

- 1) Prepare a glass of warm water around 70°C ~80°C
- 2) Put the HEXAPLAST-LITE into the warm water for about 10~15 seconds until the color becomes clear.
- 3) Soaked the HEXAPLAST-LITE get a ductility that can be curved by a little force, also maintain a steady elasticity.
- 4) After taking the HEXAPLAST-LITE away from the water, use a dry towel to remove the water.
※ Notice: Must remove hot water before placing HEXA-LITE in contact with the patient's skin to avoid possible burns.
- 5) Applies the HEXAPLAST-LITE to the nose and cover with gauze, press gently the HEXAPLAST-LITE to fold it around the nasal bone.
- 6) HEXAPLAST-LITE is formed suit to the nose, depending on the temperature descent while retaining the shape of the nose is hardened.
- 7) Once the HEXAPLAST-LITE is applied at the place, make it sure with a dressing tape. Leave HEXAPLAST-LITE at the place for seven to ten days then removes and discards it afterward.

Precautions

- 1) When using the product, temporary rash can appear on sensitive skin.
- 2) Do not use the product for other purposes.

Storage Conditions or Method

Avoid direct sunlight and store the product at room temperature

*This product is a non-sterile medical device.