

Surgiform®

Innovative Surgery Products

SAF-T-VAC®

Smoke and Fluid Evacuator

Item Number:
ST9000S

LOT

EXP

MFG

CONTENTS:

(1) each SAF-T-VAC (1) each Retention Helix (2) each Stay Strips

INDICATIONS FOR USE

The SAF-T-VAC is an accessory device used to provide an air suction path for the reduction of gases, body fluids, and smoke associated with the use of an electro-cauterization system. This device is meant to be an add-on device to many currently designed and marketed electro-cauterization systems, and standard operating room suction systems. The SAF-T-VAC is packaged as STERILE for SINGLE USE ONLY. Any use not described in these directions is not recommended.

CONTRAINDICATIONS FOR USE

Although the device itself presents no contraindications, the professional user should be thoroughly familiar with the operation parameters of the electro-cauterization system being used and the precautions for use of such systems.

DIRECTIONS FOR USE

The SAF-T-VAC is designed for placement over the end of the electro-cauterization pencil and electrode tip assembly. When slipped over the end, the electrode should extend approximately 2mm to 3mm beyond the SAF-T-VAC. Slip the retention helix over the handle to secure the SAF-T-VAC to the pencil assembly, if required. The enclosed stay strips may be used in conjunction with or in lieu of the retention helix if additional stability is desired. The adapter at the other end of the SAF-T-VAC is connected to an operating room suction system. Verify the operating room suction system is turned on and operational.

WARNING:

DO NOT USE AN ELECTROCAUTERIZATION SYSTEM IN THE PRESENCE OF FLAMMABLE GASES, OBJECTS, AND/OR OXYGEN ENRICHED ENVIRONMENTS.



SINGLE USE ONLY



Do Not
Re-sterilize

Rx ONLY (USA)

STERILE EO


Sterilized Using Ethylene Oxide



Consult Instructions for Use



Do Not Use if Package is Open or Damaged

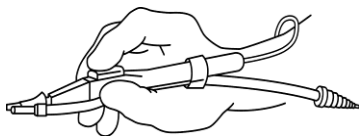

Not made with natural rubber latex.

Distributed by:

Surgiform®

1566 Whiting Way
Lugoff, SC 29078
803.462.1714 phone
803.462.1743 fax
866.225.5785 toll-free
www.Surgiform.com

Patent No. D426,883

Manufactured by: 
Surgical Technologies
Laboratories, Inc.
1566 Whiting Way
Lugoff, SC 29078
803.462.1714 phone
803.462.1743 fax
866.225.5785 toll-free

Surgiform & Saf-T-Vac are registered trademarks of Surgiform Technology, LTD.

53-0001-01

MADE IN USA